

# Antibiotic disc DISPENSER



يوزع حتي مضادات حيويه  
علي طبق بتري دش ٩ سم



**EASYlab**  
laboratory supplies



**EASYlab**  
laboratory supplies

Susceptibility test  
in one step operation

# Antibiotic Susceptibility Disk dispenser

## Specification :

- 9 Places for antibiotic disk cartridge.
- Designed for 90 mm rounded plates.
- Volume of media in plates from 3 to 6 mm depth.

## Instructions for use:

- Enter your disc cartridge in its specific hole.
- Put cartridges locker to fix cartridges in their place.
- Make sure that locker is well placed.
- Put your petri dish in its specific place.
- Just press dispenser key firmly.

## precautions :

- Be sure to keep your dispenser stainless inferior surface sterile.
- You can keep your dispenser with disks locked in refrigerator (2-8 c), after putting it in its well sealed case.